

m|devices®

IV052402

Triple Lumen Standard Bore
Extension Set 15cm

3 Lumens with 3 NeutralSite™ and
3 Slide Clamps

 INSTRUCTION FOR USE

DESCRIPTION

m|devices Triple Lumen Standard Bore Extension Set allows multiple IV fluid administration simultaneously. This triple lumen extension set is 15cm in length and each lumen has a slide clamp and a NeutralSite™ needle free access valve. The set should always be primed prior to connecting to a peripheral catheter or access device on an IV administration set.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

CAUTION

- The integrity of the NeutralSite™ should be confirmed by the clinician before and immediately after each use.
- Glass syringes are NOT compatible with the NeutralSite™.
- Single use only.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

Ensure aseptic technique is followed at all times.

1. Prime each lumen sequentially by closing two of the slide clamps on the catheter lumens you're not currently priming.
2. Swab the NeutralSite™ port with a sterile alcohol swab and attach a pre-filled syringe or IV administration set filled with sterile fluid to the NeutralSite™ port.
3. For NeutralSite™ access: Insert the syringe with a straight motion, then push and twist a quarter turn clockwise, or until it locks, to secure the connection.
4. Push the fluid through the open lumen until all air is expelled from the tubing and reaches the male luer connector.
5. Close the slide clamp on the primed lumen and repeat steps for the two remaining lumens, ensuring all air is removed from the set.
6. Remove protective cap from male luer connector and connect into access device, securing into place by engaging rotating collar.


PRECAUTIONS


- The security of all connections should be checked once established and monitored during use to prevent disconnection.
- Do not use a needle or blunt cannula with NeutralSite™. Clean the surface of the NeutralSite™ with an approved disinfectant as per facility protocol for at least 30 seconds, before and immediately after each use to remove any residual medication/blood from the surface.
- Flush the NeutralSite™ after every use or as per facility protocol.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801



 **BQ Plus Medical Co., Ltd.**
No. 18, Che Ye Road
Che Dun Town, Songjiang
201611 Shanghai, China

 **Prolinx GmbH**
Brehmstr. 56
40239 Duesseldorf
Germany



Rev: New

Rev: New