

m|devices®

IV070004

Vented Spike Adapter
with Needleless Access Site
10cm

 INSTRUCTION FOR USE

DESCRIPTION

Designed for safe and efficient fluid transfer, the m|devices Vented Spike Adapter enables the transfer of fluids from bags, vials or bottles without the need for needles. The needle-free access site minimises the risk of needle stick injuries, promoting user safety.

FOR USE BY A QUALIFIED CLINICIAN. THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.

 CAUTION

- Single use.
- DO NOT re-sterilise.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight. Handle with care.
- STERILE (EO), DO NOT use if the package or product has been damaged or contaminated.
- EU Notice: any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and /or patient is established.

STEPS

1. Maintain aseptic technique while removing the product from its packaging.
2. Remove the protective cap from the spike and insert it into the fluid bag or container. If the container is rigid, open the vent on the spike.
3. Suspend the bag or container.
4. Clean the needleless access site with a regulatory-approved disinfectant and insert a syringe using a push and clockwise turn motion.
5. Draw back the plunger on the syringe to prime the line and withdraw the required amount of fluid.


PRECAUTION


- The security of all connections should be checked once the set is established and monitored during use to prevent disconnection.

m|devices

27 Llewellyn Avenue
Villawood NSW 2163
Australia
T +61(0)2 8718 2800
F +61(0)2 8718 2801



 **BQ Plus Medical Co., Ltd.**
No. 18, Che Ye Road
Che Dun Town, Songjiang
201611 Shanghai, China

 **Prolinx GmbH**
Brehmstr. 56
40239 Duesseldorf
Germany



Rev New